

Skinnia Wounds



Apósito cicatrizante en spray

Indicación

- Heridas superficiales
- Cortes
- Rasguños
- Rozaduras en caballos

Composición

Etanol, Polímeros filmógenos, Mentol.

Modo de acción

Al ser aplicado, el alcohol ejerce una acción desinfectante inmediata, evaporándose en cuestión de segundos y favoreciendo así la formación del film protector sobre la piel. Además, gracias a la excelente combinación de excipientes, favorece el proceso de cicatrización acelerando así la curación de la piel dañada.

Modo de empleo

Agitar bien el envase antes de usarlo y pulverizar sobre la piel. Pulverizar generosamente en el área afectada 2 o 3 veces por día hasta que la zona recupere su apariencia normal.

Especies de destino

Perro, gato y caballo.

Formato

Botella con pulverizador de 30ml y 100ml.

Precauciones

Mantener fuera del alcance y la vista de los niños. Evitar contacto con ojos y mucosas.

Principales ventajas



Envase pulverizador
Fácil aplicación



Película transparente
Vigilancia directa



Sabor amargo
Evita el lamido



Aislamiento de la herida
Alta permanencia



Barrera transpirable
Evita la maceración

CASO DE USO

Nombre: CURRO | **Raza:** MESTIZO | **Edad:** 14 AÑOS | **Estado:** HERIDA EN LA PIEL



Día 0



Día 3



Skinnia Wounds



Healing sprayed-on film



Indication

- Superficial wounds
- Cuts
- Scratches
- Scratches on horses

How to use

Shake the container well before use and spray onto the skin. Spray generously onto the affected area 2 or 3 times a day until the area regains its normal appearance.

Composition

Ethanol, Film-forming polymers, Menthol.

Target species

Dog, cat and horse.

Mode of action

When applied, the alcohol has an immediate disinfectant action, evaporating in a matter of seconds and thus promoting the formation of a protective film on the skin. In addition, thanks to the excellent combination of excipients, it promotes the healing process, thus accelerating the healing of damaged skin.

Format

30ml and 100ml spray bottle.

Cautions

Keep out of the reach and sight of children. Avoid contact with eyes and mucous membranes.

Advantages



Spray bottle
Easy application



Transparent film
Direct surveillance



Bitter taste
Prevents licking



Wound isolation
High permanence



Breathable barrier
Prevents maceration

STUDY CASE

Name: CURRO | **Breed:** MIXED | **Age:** 14 YEARS | **Condition:** SKIN WOUND



Day 0



Day 3